NANOGEN AKTIV INSTRUCTIONS FOR USE (English)

Description: Nanogen Aktiv is a bio-cellulose nano-fibred membrane which is derived from a unique plant extract. The product provides a nano-structured membrane, which provides an Extra Cellular Matrix (ECM) which aids cellular migration across the wound; it also provides a variety of nutrients, natural acids, enzymes and vitamins to the wound bed, which are proven to accelerate wound healing. It requires the use of a secondary dressing (E.g. GentilFoam) to avoid wound dehydration.

Indications : Suitable for use on partial and full thickness wounds such as Leg Ulcers, Pressure ulcers, Diabetic ulcers, trauma wounds, complex surgical wounds and Epidermolysis Bullosa. Suitable for application on all stages of wound healing, except wounds with more than 50% black necrotic tissue. Nanogen Aktiv is a 100% natural antimicrobial product and can be used with confidence on infected and critically colonized wounds. Excellent at 'kick-starting' chronic wounds that are stuck in the inflammatory phase of healing, balancing the wound which then allows endogenous healing to occur.

Contraindications: None
Method of Usage: Choose the appropriate size, the membrane can overlap the wound margin without impacting on healthy tissue, but must cover the whole wound, cutting of the dressing is permitted. Once the dressing is placed on the wound, the exudate should activate the membrane. If necessary, the membrane can be soaked in sterile saline prior to application. Can be used in conjunction with Nanogen Aktigel for optimal results.

Frequency of use: The membrane can be left in situ for seven days. If the secondary dressing needs changing, take care not to disturb the membrane during removal / application of the new dressing.

Removal: The membrane can be removed in one piece. If necessary soak with sterile saline prior to removal.

Warnings: 1.Please make sure that wound bed is clean prior to application. 2. The active ingredients will be absorbed into the wound bed within 3 days. 3. A suitable secondary dressing (E.g. GentilFoam) must be used.

Storage: Nanogen Aktiv dressings should be kept in a well-ventilated, dark environment with less than 80% relative humidity, between 10°C and 40°C temperature.

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Genadyne Biotechnologies Inc.