Aldanex®

Skin & Incontinent Cleanser

Aldanex® Skin & Incontinent Cleanser

Aldanex® Skin & Incontinent Cleanser is a gentle, pH balanced, non-aerosol, no-rinse, fragrance-free cleanser formulated to moisturize as it cleanses the patient’s skin. No animals were used in the safety testing of Aldanex® Skin & Incontinent Cleanser.

Indications: For cleaning intact or injured skin associated with common severe or chronic diarrhea or urinary or fecal incontinence, and for removing dried blood, post-operative antiseptic solutions and other hard to remove debris from the patient’s skin. Aldanex® Skin & Incontinent Cleanser is also effective for removing skin protection barriers such as Aldanex® Skin Protectant.

Warnings: FOR EXTERNAL USE ONLY. Avoid contact with eyes. If this happens, rinse thoroughly with water. If condition worsens or does not improve within 7 days, consult a physician. Avoid use on persons with known allergic sensitivity to any ingredient listed. Keep out of reach of children.

Precautions: Do not use past expiration date. Do not use if product is damaged.

Directions for Use: If using a new bottle, turn nozzle to the “on” position and prime the sprayer by squeezing the pump trigger once or twice before initial use. For foaming action, pull the nozzle tip forward; for spray, push the tip back.

FOR INCONTINENT CLEANSING. Spray Aldanex® Skin & Incontinent Cleanser onto the affected area and wipe off. Rinsing or moisturizing is not necessary. Aldanex® Skin & Incontinent Cleanser can be used prior to the application of an appropriate skin protectant product such as Aldanex® Skin Protectant.

Ingredients: Purified Water, Disodium Cocoamphodiacetate, Glycerin, Poloxamer 188, Cocamidopropyl PG – Dimonium Chloride Phospate, Xanthan Gum, Disodium EDTA, Benzethonium Chloride, Citric Acid.

Storage: For best results, store at room temperature. Avoid extreme heat. Do not freeze.

CE	REP
Emergo Europe	
Molenstraat 15	
2513 BH, The Hague	
The Netherlands	
Tel: (+31) 70 345-8570	

	BioMed Laboratories, LLC
8181 Eastpoint Dr., Suite 500	
Dallas, TX 75227 U.S.A.	
www.biomedlabs.com	
REV. 07/15	
401-IN02	

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