



INNOVATION IN (BIO)ACTIVE WOUND MANAGEMENT

# Aldanex<sup>®</sup>

Skin Protectant

## Application

**IFU Aldanex:**  
**Use less product**  
**for more economy**  
**and optimum result**



**New tests in the US have occasioned a slight modification of the Instructions For Use (IFU) relating to BiologiQ's silicone based barrier cream Aldanex.**


The tests show that the ideal dosage for a skin surface measuring approximately **60 square centimetres** (up to 8 x 8 centimetres) is a **pea-size 'droplet'** of Aldanex.

### Apply a thin layer of Aldanex

Thoroughly cleanse the entire skin area with a suitable cleanser. Wash or clean hands.

Squeeze a small quantity of Aldanex from the tube onto a finger and apply this in the middle of the skin area you want to treat or protect. Gently and evenly distribute the Aldanex onto the skin. Do this in a circular motion from the centre outwards (avoid zig-zagging).


### Superior adhesion

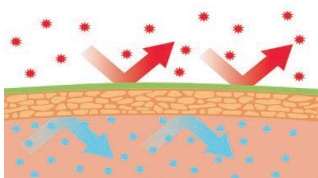

This small quantity is sufficient to create an even barrier across the entire skin area.

Precisely the product's strong adhesive qualities enable Aldanex to be distributed evenly across the skin.

**Use and store Aldanex at room temperature. Leave the tube standing upright, when not in use.**

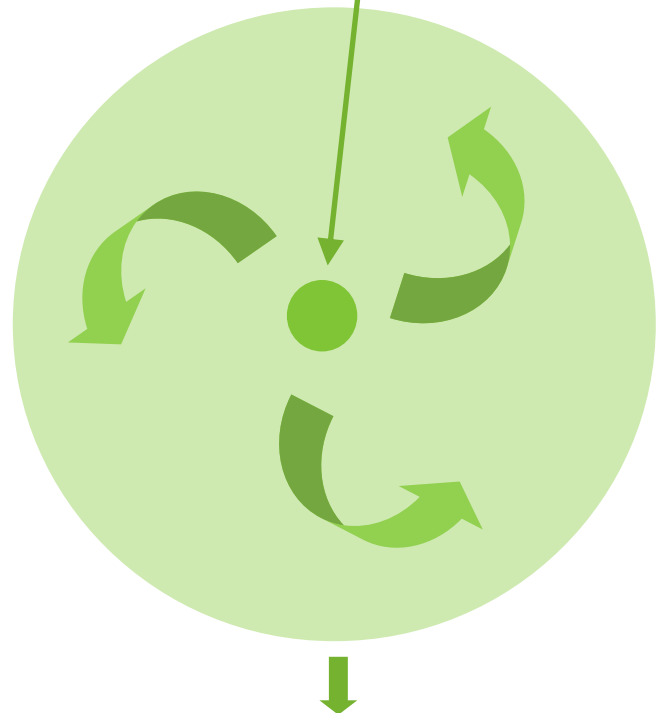

**A pea-size 'droplet' of Aldanex is the ideal dosage for a skin surface measuring approximately 60 square centimetres. (up to 8 x 8 centimetres)**