

[TEXTUS] heal®

(GB)

HYALURONIC SPRAY TO IMPROVE WOUND HEALING

[TEXTUS]heal® is a hyaluronic spray to enhance wound healing, especially for stagnating or granulating wounds and to accelerate the healing process for fresh skin lesions.

FIELD OF APPLICATION

[TEXTUS]heal® is intended for the external, local treatment of wounds starting in the granulation phase and of external skin defects such as for example: ulcera cruris, decubitus, diabetic foot ulcer, first and second degree burns, skin graft areas, abrasions, etc.

An important condition for the use of [TEXTUS]heal® is a non-infectious and clean wound.

DOSAGE & APPLICATION

1. Two or more spray applications with each dressing change, at regular dressing change intervals.
2. A triple recontamination protection in the spray bottle allows application on multiple patients. Avoid wound contact with the spray bottle by maintaining at least a 15 cm spraying distance.
3. Spray [TEXTUS]heal® directly on the non-infectious, clean wound and cover with a sterile dressing such as for example Aquafilament dressing [TEXTUS]balance®. A possible wetting of the surrounding area of the wound resulting from this has no harmful effects.
4. Taking the surrounding wound area under consideration fixate the treated wound area and the dressing semi-permeably (e.g. with [TEXTUS]biofix®) to prevent drying-up.

COMPOSITION

Main ingredient: 0.2 % hyaluronic acid-sodium salt in ringer's solution. Free of preservatives.

CONTRAINDICATION

Infected and unclean wounds (e.g. distinct coverings and / or necrosis).

Simultaneous use of antiseptics.

SIDE EFFECTS

Up to now no side effects are known since [TEXTUS]heal® is of synthetic origin and thus free of any animal content.

NOTE

The content is sterile as long as the package is intact.

For expiry date see imprint of package; use for a maximum of 12 weeks after first application. Store dry at room temperature.

SIZE AND PACKAGING

15 ml 1 pc./VE
15 ml 3 pc./VE

MANUFACTURER

biocell

Gesellschaft für Biotechnologie mbH

Oberstaat 6

D - 51766 Engelskirchen

Fon +49-2263-92917-0

Fax +49-2263-92917-10

www.biocell.de

info@biocell.de

Certified according to DIN EN ISO 9001 / DIN EN ISO 13485

Status of information: 2014-09

