Aldanex®

Skin Protectant

Aldanex is a smooth, viscous, greaseless, fragrance-free, topical silicone-based ointment for skin protection. It is also suitable for prevention of skin breakdown from environmental exposure. Aldanex adheres well to moist or dry skin and is easy to remove.

Indications:

Aldanex® Skin Protectant is indicated for the prevention and care of incontinence-associated dermatitis (IAD), pressure ulcers and other moisture lesions.

Aldanex® Skin Protectant:

- Prevents the development of IAD, moisture lesions, pressure ulcers, and intertrigo.
- Promotes skin repair by creating ideal conditions for natural healing.
- Protects irritated and/or damaged skin.

Warnings:

FOR EXTERNAL USE ONLY. Avoid contact with eyes. If this happens, rinse thoroughly with water. If condition worsens or does not improve within 7 days, consult a physician. Avoid use on persons with known allergic sensitivity to any ingredient listed. Keep out of reach of children.

Precautions:

Do not use past expiration date. Do not use if product is damaged.

Directions for Use:

Thoroughly cleanse the entire skin area with a suitable cleanser. Wash or cleanse hands. Remove ointment from tube and apply a thin coating of Aldanex by rubbing gently onto the skin. Repeat after each sanitary change or as often as required.

Ingredients:

Dimethicone, Polyethylene Glycol 8, Polyethylene Glycol 80, Glycerin, Propylene Glycol, Povidone, Stearyl Alcohol.

Storage:

For best results, store at room temperature. Avoid extreme heat. Do not freeze.

Disposal:

Dispose of the device in accordance with Local Regulations

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