

# CINESTEAM<sup>®</sup>

## QUICK GUIDE FOR USE



### Device description:


Cinesteam<sup>®</sup> is a secondary dressing designed to reduce odour and absorb exudate. It is composed of an absorbent core and a sachet containing cinnamon powder.

### Before using Cinesteam<sup>®</sup>:

For full instructions, please refer to the Instructions For Use (IFU) included in the box. Cinesteam<sup>®</sup> must be applied over a primary wound contact layer and should not be applied directly to the wound.

### Cinesteam<sup>®</sup> application:

- 1.** Open and remove Cinesteam<sup>®</sup> from its pouch.
- 2.** Apply the white face Cinesteam<sup>®</sup> on top of the primary dressing so that it covers it entirely.
- 3.** Secure Cinesteam<sup>®</sup> in place using adhesive tape or a retaining bandage.

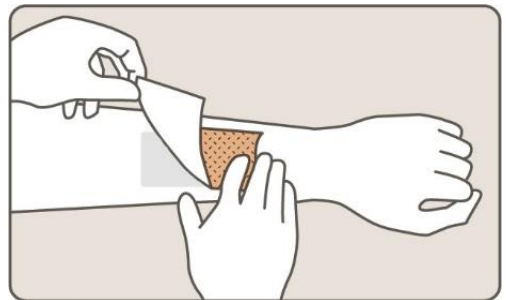

### **Cinesteam® renewal:**

Gently remove Cinesteam® and replace following the application instructions. The frequency of dressing change must be adapted to the wound care protocol in place. Cinesteam® can be left in place as long as its absorption and odour masking capacities are satisfactory (1 to 7 days). Frequency of change should be dependent on the level of exudate.

### **Precautions**

Do not use Cinesteam® as a primary dressing.

Do not cut Cinesteam® because this could induce cinnamon leakage.

Cinesteam® should not be used if its primary packaging is open or damaged or if the dressing itself is torn.

Do not use Cinesteam® together with an active charcoal dressing, which may decrease its odour masking ability

Do not cover Cinesteam® with an occlusive fastening material or dressing.

Do not eat the cinnamon of Cinesteam®.

Do not bathe or shower when wearing Cinesteam®. Do not immerse Cinesteam® in water.

Keep out of the reach of children.

### **Contraindications**

Cinesteam® should not be used on patients with known sensitivity to any of its components, in particular cinnamon.

#### **Legal mentions :**

Cinesteam® is a secondary anti-odour absorbent dressing manufactured by CEMAG Care.

Cinesteam® is intended to be used for the management of malodorous wounds.

This Class I Medical Device is compliant with the European Essential Requirements.

Read carefully the instructions before use. Ask your healthcare professional, doctor, pharmacist or nurse for more information.

[www.cinesteamcare.fr](http://www.cinesteamcare.fr)

[contact@cinesteamcare.com](mailto:contact@cinesteamcare.com)

FC 004DLI003/01-12.22